



## **OPTI-GARD® - EYE PROTECTOR**

---

LATEX FREE – SINGLE- USE

### **Rx ONLY**

Opti-Gard® is used to protect the patient's eyes from inadvertent contact during medical procedures.

### **DIRECTIONS**

1. Remove paper backing.
2. Position over eyes ensuring that the eyes are within the openings of the lenses.
3. Press edges firmly and carefully for a tight bond.
4. Remove slowly and gently when no longer needed.

### **PRECAUTIONS**

1. Do not use on a patient whose eyes do not fit within the openings of the lenses.
2. For patients with a small face use Small Opti-Gard® P/N 28200.
3. This device is not designed to bear weight. Do not place objects on the eye protector. Use a head support that prevents pressure from being put on the Opti-Gard® when a patient is in the prone position.
4. Frequently check to ensure there is no pressure on the patient's eyes during the use of this device and that the eyes remain within the openings of the lenses.

### **WARNINGS**

1. **Compression of the eye protector on the eyes of the subject can lead to serious adverse effects, including blindness.**
2. **Some erythema may result from skin contact with the materials used in this device, or from use during a long procedure.**
3. **Store product at 40° to 80° F.**