



ProneView® Spinal Surgical Frame Positioning Pack

Catalog Number D28630

Positioning pack to be used with ProneView® Protective Helmet System

Enclosed Items:

- 1 ea. ProneView® Cushion Insert
- 1 ea. Base Cover
- 1 ea. Cross Bar Pad
- 2 ea. Support Pad Covers
- 2 ea. Arm Support Pads (1 each # M28604 Right / 1 each # M28606 Left)

CUSHIONS ARE NOT MADE WITH NATURAL RUBBER LATEX

Suggested Procedure:

Note: Open all vacuum packages 60 minutes prior to use.

1. Open the ProneView® Cushion Insert and follow instructions provided.
2. Disengage the pad supports from their distal mountings by pushing in and up at the foot end of the surgical frame. See Figure "A"
3. Locate the support pad covers, and slide over the pads ensuring the elastic strip is at the bottom of the support. To avoid tearing, pull from the elastic strip. See Figure "A"
4. Repeat this step for the other pad.
5. Locate the disposable base cover, and place on base with the adhesive tabs facing down. Remove the adhesive liner and affix the adhesive tabs to the side edge of the base on all four corners. See Figure "B"
6. Replace the pad supports to their distal mountings; location is at the foot end of the frame.
Note: When replacing the pad supports, ensure the supports are under the lip provided in the base of the frame.
7. Locate the cross bar pad; place on the bar at the head end of the frame. See Figure "C"

